


# Megacode on SmartMan

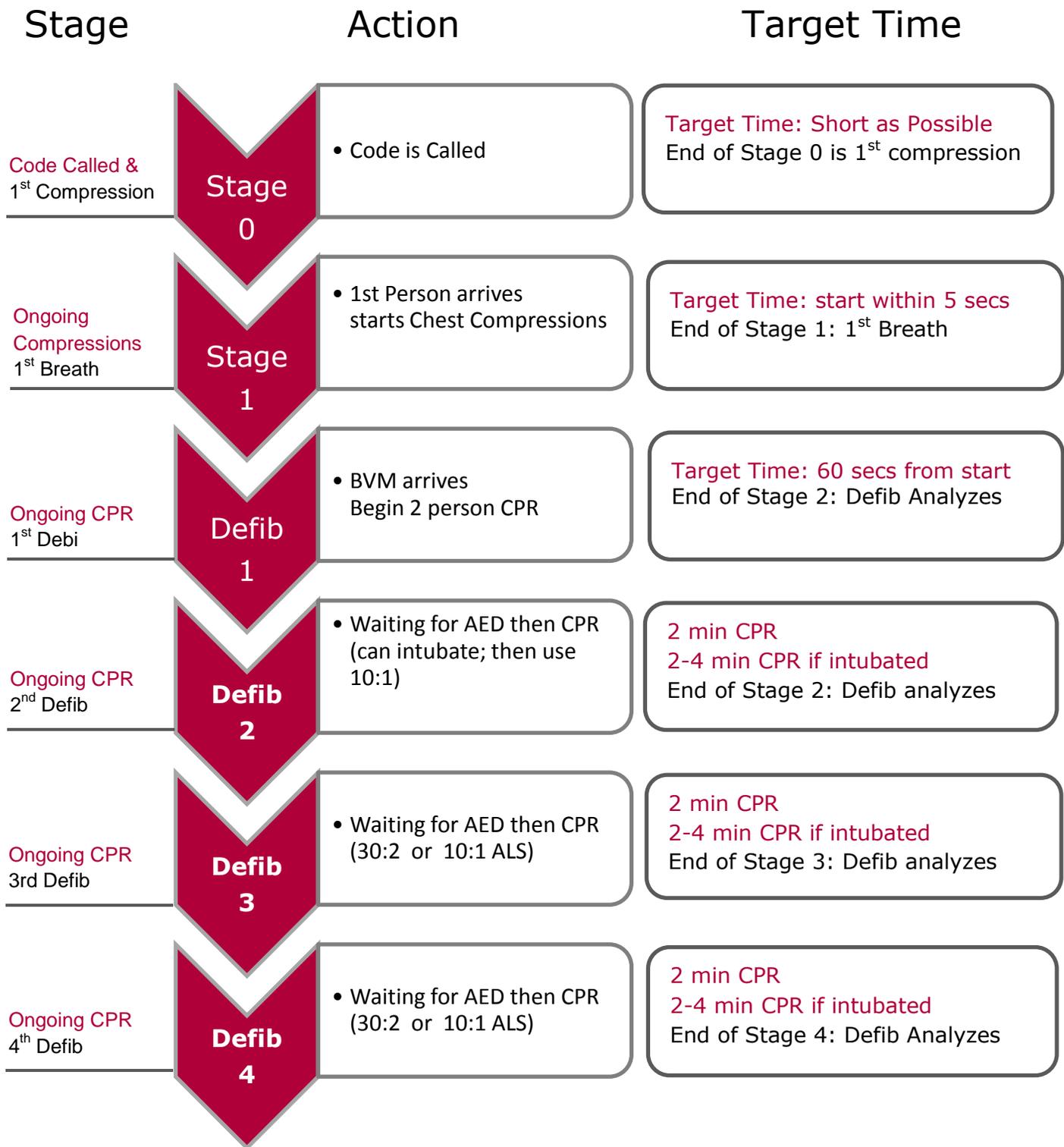

Stages continue until **STOP** Button is clicked. Maximum time is 20 minutes.

Click Buttons to indicate giving epinephrine, vasopressin, amiodarone, to indicate IV established, indicate now intubated

# Megacode on SmartMan

## Options and Variations

### Megacode AHA (CPR following AHA 30:2)

End of Stage 1 (After first use of AED) if running AHA protocol there are 3 choices for how CPR is performed:

- 30:2
- 10:1 BLS (Low Volume 300ml – 400ml vent provided every 10<sup>th</sup> breath)
- Click Intubate Button and go to 10:1 ALS (Normal Volume)

### Megacode ROC (CPR Following ROC 10:1 BLS Low volume)


CPR begins with 10:1 BLS low volume

End of Stage 1 (After first use of AED) there are 2 choices for how CPR is performed:

- 10:1 BLS (Low Volume 300ml - 400ml)
- Continue with 10:1 BLS (Low Volume 300ml – 400ml vent provided every 10<sup>th</sup> breath)
- Click Intubate Button and go to 10:1 ALS (Normal Volume)

### Critical Interventions

Buttons indicate when a particular intervention takes place. The time these are performed in relation to the other actions will be shown on the Health Status Line and in the Debriefing Report

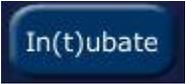Buttons	To Indicate	Alternate
	IV Established Epinephrine Vassopressin Amiodarone	Press the <b>I</b> Key Press the <b>E</b> key Press the <b>V</b> Key Press the <b>A</b> Key
	Intubate	Press <b>SPACE BAR</b>
	Defibrillator	Press the <b>D</b> key
	Stop	Press the <b>S</b> key